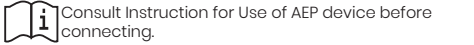
Gold Eartip Electrode

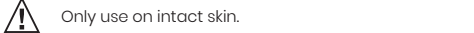
Electrodes, Neurological

Instructions for Use – UK/US

Gold Eartip Electrodes are combined ear tips and cutaneous electrodes intended for use with auditory evoked potential (AEP) measurement equipment.

-  Single use only

-  Consult Instruction for Use of AEP device before connecting.

-  Only use with alligator electrode cables compliant with DIN 42802-1 & IEC 60601-1.
-  Do not use electrodes for stimulation purposes.
-  Only use on intact skin.

This device is for use by trained professionals only. Intended patient population is children and adults.

In case of serious incident in relation to the use of the device, the incident must immediately be reported to DGS Diagnostics A/S and the local national competent authority.

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- Use otoscope to check that ear canal is not blocked, injured or infected. Do not use this device if there are any contraindications.
- Gently compress the eartip by rolling between your fingers.

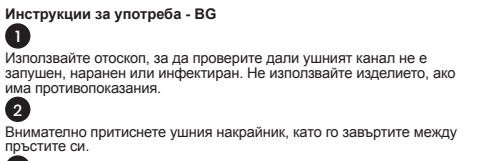
Figure 1: Correct use of otoscope

4 squeeze the tabs of the clip and release on the eartip

- Push the tube end of the eartip fully into the sound tube of your transducer
- Insert the eartip fully into the ear canal as shown.

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